

# EASYBOOT MINI APPLICATION

## Sizing the Easyboot Mini for your Horse

- » Take hoof measurements immediately following a trim.
- » Follow the steps on the back of this guide to select the correct size. Correct sizing is imperative for performance.
- » The Easyboot Mini comes in various sizes to fit a range of miniature horse, pony and foal hooves.
- » Easyboot Mini sizing allows for natural hoof growth throughout the trim cycle. Do not round up.
- » If you have concerns about sizing or fit, please contact our Product Specialists at 800-447-8836.
- » If you intend to use the Easyboot Mini boots over steel shoes\*, take your hoof measurements after the horse is freshly shod.

*\*Using our hoof boots over metal shoes voids the EasyCare Guarantee.*



## Fitting the Easyboot Mini on Your Horse

1. Fully open the upper to ensure that no hook and loop areas are touching.
2. Fold down the back portion of the boot.
3. Facing the rear of the horse, pick up the foot and slip it into the boot, toe first. Place the sole of the hoof firmly down into the Easyboot Mini. Pull the rear of the boot up around the heel bulb.
4. Place the foot on the ground to a weight bearing position. Lift the opposite leg to assure the hoof is fully weight bearing.
5. Pull the rear flaps around the pastern and secure the hook and loop.
6. Confirm that the hook and loop is centered on the pastern bone. The Easyboot Mini is now securely in place.



1.



2.



3.



4.



5.



6.

## Easyboot Mini Fitting Tips

- » Easyboot Mini boots are not left/right specific and will fit front or hind hooves.
- » Incorrectly sized Easyboot Mini boots can cause chafing. Correct size selection will ensure maximum performance and longevity of the boots.
- » Give your horse time to adjust to wearing the Easyboot Mini. Start with conservative use to allow for break in.
- » Gradually increase the duration of use. If being used for therapy, check for chafing frequently.

## HOW TO MEASURE FOR THE EASYBOOT MINI

### Easyboot Mini

The Easyboot Mini has been designed specifically for miniature horses, ponies and foals. Quick and easy to apply, this boot is the ultimate option for tiny hooves.

The lightweight urethane blended sole will allow for traction and protection on any surface. The durable and supple upper keeps the boot secure without interfering with the natural gait.


This is the ideal boot for light driving, therapy and providing traction.

### Unique Features of the Easyboot Mini

- » Urethane blend of materials yields a flexible and long wearing sole.
- » Upper folds flush with the base of the boot for ease of application.
- » Easy to adjust upper to fit different pastern sizes and angles.
- » The Easyboot Mini is the smallest molded hoof boot in the world.

The Easyboot Mini is a first class option for horses who encounter diversity of terrain. The Easyboot Mini will accommodate a Comfort Pad to be used as a therapy boot.

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

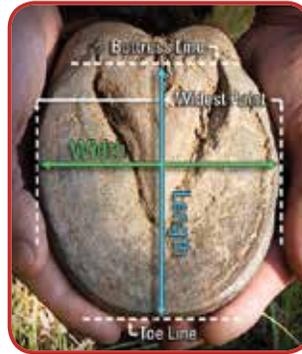

Size	Width in inches	Length in inches
Mini	44-49	49-54
Mini 1	50-55	55-64
Mini 2	56-65	65-72
Mini 3	66-76	73-86

### Join the Conversation

For tips and tricks follow EasyCare:



### A product of EasyCare Inc.

EasyCare Guarantee

800.447.8836 | easycareinc.com

All products should be used in accordance with the instructions in this application guide. EasyCare Inc. hereby disclaims responsibility for the misuse of any product and for losses incurred through the use of any product described in this guide, on EasyCare Inc. product packaging, or on easycareinc.com. EasyCare Inc. products are not intended to diagnose or cure any medical condition.

# Easyboot MINI

## APPLICATION GUIDE

Designed specifically for miniature horses, ponies and foals



**THE SMALLEST MOLDED HOOF BOOT IN THE WORLD**



**EasyCare Inc.**  
The Ultimate in Hoof Protection