

# EasyShoe *PERFORMANCE N/G*

## Application Guide: INSTALLING STUDS



Developed by *Ernest Woodward*

### MATERIALS REQUIRED:

- 1 X pair EasyShoe N/G
- 1" wood bit
- 7/16" metal bit
- x4 3/8 x 16 Blind Nuts (T Nuts)
- EasyShoe Bond or Vettec Adhere
- Sharpie Pen
- Hammer

### APPLICATION:

1. Mark an "X" on the inside of the shoe that leaves a 1/2" radius from the metal insert and the edge of the EasyShoe.
2. Using the 1" wood bit, drill approximately half way through the sole of the EasyShoe.
3. Using the 7/16" metal bit, drill out the center of the hole through the entire sole of the EasyShoe.
4. Set the blind nut into the hole and drive it into the EasyShoe with a hammer.
5. (Optional) Plug the threads with cotton or paper towel and backfill the hole with EasyShoe Bond or Vettec Adhere to create a smooth surface for the foot to stand on.

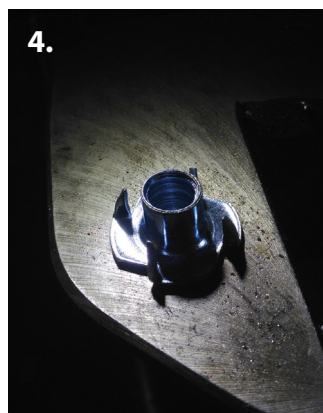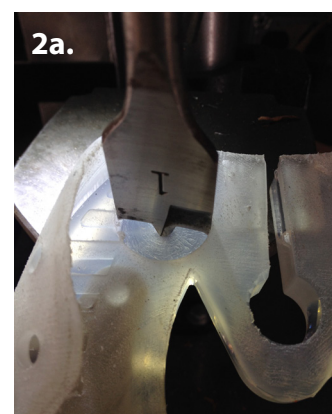

# EasyCare

800.447.8836  
easycareinc.com