

Easyboot Transition Application Guide

The Seamless Integration of Flexibility and Durability

The Easyboot Transition offers an innovative combination of shock absorption and support never before seen in a hoof boot. The lightweight upper protects the hoof and pastern joint from trail hazards. Reversible hook and loop straps at the front and back secure the boot on the hoof and allow for precise fit adjustments. At the core of the boot, a polyurethane bumper offers pliability and resilience. The flexible bumper and hardy outsole are molded together to form a supportive midsole for the hoof to stand on. Depressions within the outsole structure are filled with the softer midsole material to create an integrated cushion. At its base, there is a trail-proven tread pattern that excels in multiple disciplines.

The Easyboot Transition is available in eight sizes that accommodate many hoof shapes and angles. This is an ideal hoof boot for pleasure riding (less than 25 miles per week) and is also suitable for therapeutic applications.

The Easyboot Transition features the latest in technological design:

- Leather and soft weave fabric construction protects the hoof from trail hazards.
- Breathable mesh upper.
- Reflective details increase visibility in low light.
- Bumper adds flexibility to the core of the boot.
- Industrial strength hook and loop straps allow for multiple adjustments.
- Flexible bumper and hardy outsole seamlessly molded together for additional shock absorption.
- Fabric & synthetic outsole resists wear and increases flexibility.
- Semi-aggressive trail-proven tread.
- Sold in singles.

Correct Application of the Hook and Loop Straps

EasyCare recommends attaching the hook and loop straps so they face out when applied. To change the direction of the strap, simply follow the instructions below.

1. Remove the strap from under the rubber keeper, open the strap and remove it from the first D-ring. Pull the front strap completely through the center leather loop.
2. Moving the rubber keeper out of the way, undo the remaining strap from itself and remove the strap completely from the second D-ring.
3. For the front strap, start threading from the rear of the boot. Thread the hook and loop strap through the D-ring that will face out and fold the strap against itself so that the hook and loop material attaches. Move the rubber keeper back towards the D-ring.
4. Thread the entire strap through the center leather loop (front strap only) and through the second D-ring. Fold the strap back on itself so the four hoof logo is visible. Move the rubber keeper back over the end of the strap.
5. For the rear strap, start threading from the front of the boot. Follow steps 3 and 4.



Easyboot Transition Application

Fitting the Easyboot Transition on Your Horse

1. Unhook the hook and loop straps at the front and rear of the boot.
2. Completely fold down the rear section of the soft weave upper so it is flush with the back of the bumper.
3. Facing the rear of your horse, pick up the hoof and slip it into the boot, toe first.
4. Using technique as opposed to force, gently twist the boot until the hoof is completely in the boot and free of any obstruction.
5. Pull the rear section of the soft weave upper completely over the heel bulb. Place the hoof on the ground to a weight-bearing position. Verify that the boot is fully seated on the hoof.
6. Starting at the front of the boot, secure the hook and loop strap as tightly as possible. Use the rubber keeper at the end of the strap to keep the contact secure. The collar of the boot should be snug all the way around the pastern bone.
7. Secure the hook and loop strap at the rear of the boot, ensuring the rear section of the fabric weave upper is between the horse's pastern and the strap. Use the rubber keeper at the end to secure the strap.
8. The Easyboot Transition is now securely in place.

Increase the lifespan of your boots by hosing them down after each use with cold water and allowing them to dry in a well-ventilated area out of direct sunlight.



Easyboot Transition Sizing & Fitting Tips

- Easyboot Transition boots are not left/right specific. EasyCare recommends attaching the hook and loop straps so they face out when applied.
- Riding your horse in Easyboot Transition boots that are not sized correctly can lead to chafing. Selecting the correct size will ensure maximum performance and hoof boot durability.
- Allow your horse time to become accustomed to wearing the Easyboot Transition boots. The first time you use the boots, start by just walking out.
- Gradually increase the duration and distance of your rides to ensure your horse becomes accustomed to the new hoofwear. Too much too soon can lead to chafing.

Sizing the Easyboot Transition for Your Horse

Always take hoof measurements immediately following a fresh trim.

Follow the steps on the back of this guide to select the correct size. Feel free to contact a member of our customer service team for assistance.

Easyboot Transition hoof boots are available in 8 sizes. Important: Easyboot Transition boot sizes are not comparable to steel shoes sizes. Please consult the size chart for sizing the Easyboot Transition for your horse.

Easyboot Transition sizing allows for natural hoof growth between trimmings.

If you intend to use Easyboot Transition boots over metal shoes*, take your hoof measurements after the horse is freshly shod.

**Using our hoof boots over metal shoes voids our 90-day warranty.*



EasyCare Inc.
30-Day Money-Back Guarantee

We back all of our products with a money-back guarantee. If you are not completely satisfied with a product for any reason, you may return it for a refund of the purchase price (excluding shipping and handling) within 30 days of purchase. To qualify, all products must be returned to place of purchase and have a dated receipt. Offer not valid for EasyCare bargain bin or factory seconds. If the product was purchased from EasyCare, contact us at 800-447-8836 to receive return authorization and an RMA number. All returns must have an RMA number clearly written on the outside of package. Returns without an RMA number will be refused.

EasyCare Inc.
90-Day Warranty

EasyCare guarantees the quality and workmanship of all EasyCare products for 90 days from date of purchase. While this warranty is very broad in nature, it does not cover wear (including sole breakthrough when worn over horseshoes), misuse, abuse, factory seconds, bargain bin items, or boots purchased as used. Products returned for evaluation must be washed clean, dry and free of debris, dirt, sand and manure. Damage inflicted by omnivorous horses, rodents or other unusual abuse is not covered. Customers in the United States should contact EasyCare directly for all warranty issues at 800-447-8836. International customers should contact the dealer where the product was purchased.

Join The Conversation


For up to the minute tips & tricks follow EasyCare:

 Facebook facebook.com/easyboot  The EasyCare Blog blog.easycareinc.com

All products should be used in accordance with the instructions in this application guide. EasyCare Inc. hereby disclaims responsibility for the misuse of any product and for losses incurred through the use of any product described in this guide, on EasyCare Inc. product packaging, or on easycareinc.com. EasyCare Inc. products are not intended to diagnose or cure any medical condition.

How to Measure for Boots

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

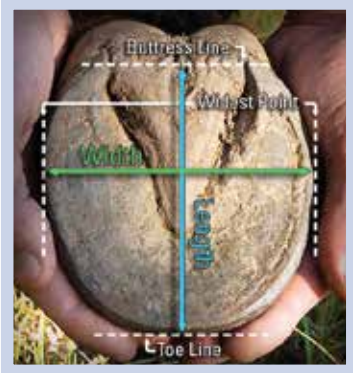

3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

Easyboot Transition Size Chart

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
000	3 1/2"-3 3/4"	3 5/8"-3 15/16"	000	89-96mm	93-100mm
00	3 3/4"-4 1/16"	3 15/16"-4 1/4"	00	97-104mm	101-108mm
0	4 1/16"-4 3/8"	4 1/4"-4 9/16"	0	105-112mm	109-116mm
1	4 3/8"-4 11/16"	4 9/16"-4 7/8"	1	113-120mm	117-124mm
2	4 11/16"-5"	4 7/8"-5 3/16"	2	121-128mm	125-132mm
3	5"-5 5/16"	5 3/16"-5 1/2"	3	129-136mm	133-140mm
4	5 5/16"-5 5/8"	5 1/2"-5 13/16"	4	137-144mm	141-148mm
5	5 5/8"-5 15/16"	5 13/16"-6 1/8"	5	145-152mm	149-156mm

Application Guide



Have Questions? Need Help? Visit easycareinc.com



EasyCare
easycareinc.com