


EasyCare Hoof Boot Stud Installation

Mark desired position of studs on sole of boot. Drill holes into sole of boot with 5/16" drill bit. Insert stud into boot from the inside. Screw the nut on the stud from the outside. Use pliers to fasten firmly. Two studs at the heel and two at the toe will give good traction.

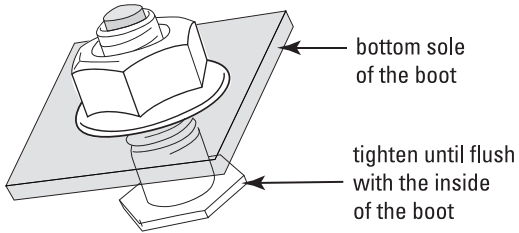

SB-STUDS - Best for all EasyCare hoof boots with the exception of Easyboots and Easyboot Epics sizes 5, 6 and 7.

SB-STUDS-L - Best for Easyboots and Easyboot Epics sizes 5, 6 and 7.

Need more help? Check out our website at easycareinc.com for hoof boot stud installation videos.