



EasyCare Inc.  
The Ultimate in Hoof Protection



**2018 Product Lineup**  
[www.easycareinc.com](http://www.easycareinc.com)

## PERFORMANCE



**Easyboot  
Glove**



**Easyboot  
Glove Soft**



**Easyboot  
Epic**



**Easyboot  
Glue-On**



**Easyboot  
Flip Flop**



**Easyboot LC**

## PLEASURE



**Easyboot  
Back Country**



**Easyboot  
Trail Original**



**Easyboot  
Trail**



**Old Mac's G2**



**Original  
Easyboot**



**Easyboot  
Mini**



**New  
Mac**

## THERAPY



**Easyboot  
Cloud**



**Easyboot  
Stratus**



**Easyboot Rx**



**Therapy Click  
System**



**Easyboot  
Zip**



**EasySoaker**

## EASYSHOE

**NEW**



**EasyShoe  
Flex**



**EasyShoe  
Performance**




**EasyShoe  
Performance N/G**



**EasyShoe  
Sport**




**EasyShoe  
Compete**




The Performance line is made for aggressive riding, long distance and speed. This line offers low profile, secure fitting hoof protection, designed to reduce interference and allow for increased training time. The Performance line offers both boots and glue-on hoof protection. Contributing to more top finishes at the world's toughest endurance races than any other brand, EasyCare's performance line has proven to hold up under the most challenging conditions.

**Our Performance line includes** the 2016 Glove, Glove Soft, Epic, Glue-On, LC and Flip Flop.



The Pleasure line-up is built with easy application and removal in mind. Any boot in this category is ideal for trail riding, as a replacement for a lost shoe or protection for a sore-footed horse.


**Our Pleasure line includes** the Trail, Trail Original, New Mac, Old Mac's G2, Back Country, Mini and Original Easyboot.



The Therapy line offer a range of innovative therapeutic boot for comfort, support, rehab and medicating. If your horse suffers from chronic lameness, founder, laminitis, thin soles, abscesses, shipping stress or any other hoof and lower limb problems, we have a boot that will provide instant and ongoing relief.

**Our Therapy line-up includes** the Cloud, Rx, Zip, EasySoaker and Stratus.


\*Most therapy boots will accommodate the Therapy Click System.




With our EasyShoe line-up you can provide constant protection to the hoof while still enjoying the benefits of a barefoot horse. The revolutionary EasyShoe is designed to allow the hoof to expand and contract, both laterally and vertically - as nature intended. Utilizing the latest polyurethane technology, the EasyShoe offers maximum durability and protection in addition to dramatically reducing concussion on hard surfaces. The EasyShoe line offers shoes that can be nailed to the hoof, glued to the hoof, or a combination of both.

**Our EasyShoe line includes** Performance, Performance N/G, Sport, Compete and all Flex options.


## Recommended Riding Distances: Miles Per Week

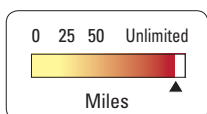

**Original Easyboot, Easyboot Epic, Easyboot Glove, Easyboot Glue-On, Easyboot Flip Flop, Easyboot LC, Easyboot Glove Soft, EasyShoe**  
(Unlimited miles per week)

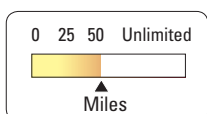






**Easyboot Back Country**  
(Up to 50 miles per week)

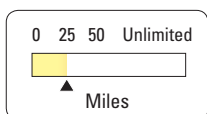

**Easyboot Trail, New Mac, Trail Original, Old Mac's G2, Easyboot Mini**  
(25 miles per week)

**The Easyboot Rx, EasySoaker, Easyboot Zip, Easyboot Stratus and Easyboot Cloud are not intended for riding.**


## Ease of Application: Easiest to Hardest

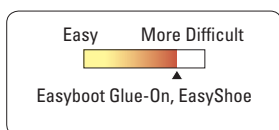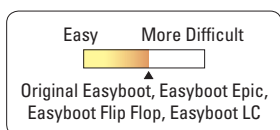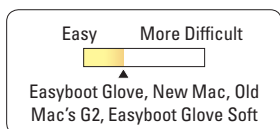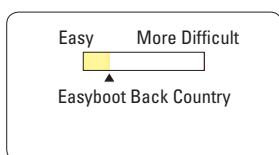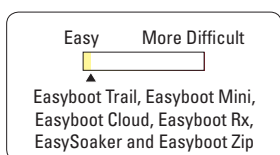

## Amount of Traction: Less to More

Easyboot Glove, Easyboot Back Country, Easyboot Cloud, Easyboot Rx, Easyboot Glue-On, Original Easyboot, Easyboot Flip Flop, Easyboot Epic, Easyboot Stratus, Easyboot Glove Soft and Easyboot LC




New Mac, Easyboot Trail, Old Mac G2 and Trail Original

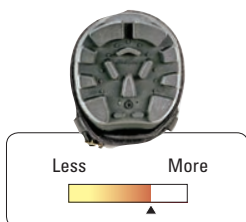

EasyShoe Performance & Performance N/G, EasyShoe Flex

EasyShoe Sport

EasyShoe Complete

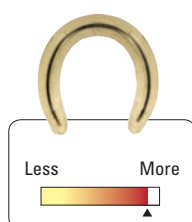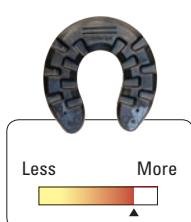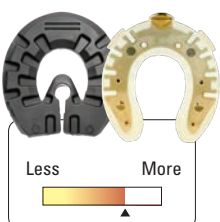


**Studs are available for increased traction on ice, mud or turf.**



## How to Measure

The biggest determining factor in EasyCare hoof boot\* and EasyShoe success is fit. Always measure as soon as possible after a fresh trim. Waiting to take measurements can result in a poor fit.

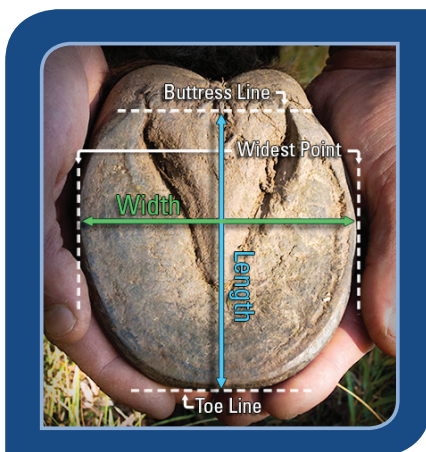

1. Measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurements with the appropriate size chart. Please note: each model has its own unique size chart.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are more than one size apart, your selected model is not recommended.\*\* Check your measurements against an alternative product in the EasyCare line—another model may accommodate your horse's hoof shape better.

*\* Hoof boots should be snug before tightening. After tightening, the boot should be a close fit around the hoof wall. You should not be able to turn or twist the boot.*

*\*\*If the hoof measurements are more than one size apart in length and width, please call a Customer Service Representative at 800.447.8836.*

## Fit Kits: Try Before You Buy



Fit Kits are available for the Easyboot Glove, Easyboot Glove Soft, Easyboot Glue-On, Easyboot Back Country, Easyboot Flip Flop, Easyboot LC and for all models of the EasyShoe.

**Call us at 800.447.8836 for additional details.**

## Have Questions?

Call 800.447.8836 or email [admin@easycareinc.com](mailto:admin@easycareinc.com). EasyCare Product Specialists are here to help you find the right boot. Check our website for information on hoof care and hoof boots.

**[www.easycareinc.com](http://www.easycareinc.com)**



## Shipping and Guarantee

### EasyCare 30-Day - Money-Back Guarantee

We back all of our products with a money-back guarantee. If you are not completely satisfied with this product for any reason, EasyCare will gladly refund your purchase price (excluding shipping and handling) within 30 days of purchase. To qualify, products must be returned to EasyCare with a completed Return Merchandise Authorization (RMA) form. Products will not be accepted without an RMA number and form. The RMA can be obtained from [www.easycareinc.com](http://www.easycareinc.com) or by calling an EasyCare Product Specialist at 800.447.8836. Customers are responsible for the cost of shipping returned product to EasyCare.

Warranty is void for boots used over steel, aluminum or composite horseshoes, for damage inflicted by omnivorous horses, rodents, or other unusual abuse.

All glue-on boots, shoes, products and accessories must be returned in their original and unopened packaging. EasyCare, at its sole discretion, may void warranty if glue-on products appear to have been removed from packaging.

Bargain Bin, factory seconds, and products purchased used are not covered under any EasyCare warranty.

**If you have purchased our product through one of our partners, then please return the product to the place of purchase for the same warranty service. EasyCare does not accept items purchased from our dealers.**

### Shipping

EasyCare orders are shipped directly from our facility in Tucson, Arizona. All orders will be shipped within two business days from the time the order is placed. If there are any back ordered items, you will be informed immediately.

### 90-Day Warranty

EasyCare Inc. offers a repair or replacement warranty on quality and workmanship for all products manufactured under the EasyCare label within 90 days of purchase. Products must be returned to EasyCare with a copy of the customer's dated receipt and a completed Return Merchandise Authorization (RMA) form. Products will not be accepted without a dated proof of purchase and a completed RMA form. The RMA can be obtained from [www.easycareinc.com](http://www.easycareinc.com) or by calling an EasyCare Product Specialist at 800.447.8836.

EasyCare staff is always willing to discuss any specific issues you may have regarding the performance of your EasyCare products. If a boot fails during warranty, it is generally a minor hardware issue. Please bear in mind that most components of EasyCare hoof boots are either replaceable or repairable. This includes wire lace, cables, buckles, rollers, screws, straps and gaiters. Most repair can be easily accomplished with simple household tools. Videos of many repairs are available on our website at [www.easycareinc.com](http://www.easycareinc.com). You may also contact us at 800.447.8836 or [admin@easycareinc.com](mailto:admin@easycareinc.com).

Products returned for evaluation must be clean and dry. Failure to return clean and dry products will result in a cleaning fee. Products that can be repaired will be repaired and returned to the customer promptly. Products that cannot be repaired will be replaced. You are responsible for shipping costs to EasyCare. EasyCare will pay for shipping back to you if product is covered under warranty. If product is not covered under warranty then we will contact you for return shipping payment. If we have not heard back from you within two weeks after contact then we will dispose of returned products.

Warranty is void for boots used over steel, aluminum or composite horseshoes, for damage inflicted by omnivorous horses, rodents, or other unusual abuse.

Bargain Bin, factory seconds, and products purchased used are not covered under any EasyCare warranty.

Easyboot Glove

A slim, form-fitting boot that is simple to apply. It features our best tread pattern yet based on 40 years of experience and the latest in technology. It's ideal for the well-maintained barefoot hoof. Power Straps are recommended for aggressive riding, hilly terrain and muddy conditions.



Success of the Easyboot Glove is determined by accurate measurements and a snug fit.

For the Easyboot Glove, you will need to measure in millimeters. Measuring in inches and converting to millimeters will render the measurement process inaccurate.

Size Chart – Normal

Size	Width in mm	Length in mm
00	96-99mm	106-109mm
00.5	100-103mm	110-113mm
0	104-107mm	114-117mm
0.5	108-111mm	118-121mm
1	112-115mm	122-125mm
1.5	116-119mm	126-129mm
2	120-123mm	130-133mm
2.5	124-127mm	134-137mm
3	128-131mm	138-141mm
3.5	132-135mm	142-145mm
4	136-139mm	146-149mm
4.5	140-143mm	150-153mm
5	144-147mm	154-157mm

Size Chart – WIDE

Size	Width in mm	Length in mm
0W	114-117mm	114-117mm
0.5W	118-121mm	118-121mm
1W	122-125mm	122-125mm
1.5W	126-129mm	126-129mm
2W	130-133mm	130-133mm
2.5W	134-137mm	134-137mm
3W	138-141mm	138-141mm

Always double check the fit using the Fit Kit before purchasing boots. Contact us for more details. 800.447.8836

Easyboot Glove Soft

The Easyboot Glove Soft is a lightweight, slim profile, form-fitting boot that is easy to apply. It was designed to fit like a second skin over the hoof without adding bulk or width. This seamless boot hugs the hoof and responds like a natural foot. Flexible and tough, the Easyboot Glove Soft adds traction and protection to the hoof, while allowing a free stride and natural breakover. Like a glove, this boot provides protection without hindering mobility.



The Easyboot Glove Soft material stretches over the hoof and clings to the wall so that debris stays out of the boot even through sandy and muddy conditions. No external hardware means that there is no need to worry about grass or sticks getting caught in the boots. The elastic cap at the rear of the gaiter fits the contours of the heel bulb while offering generous proportions of stretch and strength.

Size Chart – Normal

Size	Width in mm	Length in mm
00	96-99mm	106-109mm
00.5	100-103mm	110-113mm
0	104-107mm	114-117mm
0.5	108-111mm	118-121mm
1	112-115mm	122-125mm
1.5	116-119mm	126-129mm
2	120-123mm	130-133mm
2.5	124-127mm	134-137mm
3	128-131mm	138-141mm
3.5	132-135mm	142-145mm
4	136-139mm	146-149mm
4.5	140-143mm	150-153mm
5	144-147mm	154-157mm

Size Chart – WIDE

Size	Width in mm	Length in mm
0W	114-117mm	114-117mm
0.5W	118-121mm	118-121mm
1W	122-125mm	122-125mm
1.5W	126-129mm	126-129mm
2W	130-133mm	130-133mm
2.5W	134-137mm	134-137mm
3W	138-141mm	138-141mm

You will need to measure in millimeters for the Easyboot Glove Soft. Measuring in inches and converting to millimeters will render the measurement process inaccurate. After taking your horse's measurements, we recommend you order a Fit Kit to ensure you select the correct size for your horse. \*Please note that sizing varies for all EasyCare products. Contact us for more details. 800.447.8836

# PERFORMANCE

## Easyboot Epic

The Easyboot Epic is the ideal boot for aggressive conditions or for a horse that is difficult to keep booted. The attached Easyboot Gaiter provides extra protection to the hoof and prevents excess dirt and debris from entering the boot. It also helps secure the boot in extreme conditions such as mud, snow and water crossings.



Size	Width in inches	Length in inches
Pony	3 3/16" - 3 3/8"	3 3/8" - 3 5/8"
000	3 3/8" - 3 13/16"	3 5/8" - 4 3/16"
00	3 13/16" - 4 3/8"	4 3/16" - 4 5/8"
0	4 3/8" - 4 9/16"	4 5/8" - 4 15/16"
1	4 9/16" - 4 7/8"	4 15/16" - 5 1/8"
2	4 7/8" - 5 1/8"	5 1/8" - 5 9/16"
3	5 1/8" - 5 1/2"	5 9/16" - 5 15/16"
4	5 1/2" - 5 15/16"	5 15/16" - 6 7/16"
5	5 15/16" - 6 7/16"	6 7/16" - 6 7/8"
6	6 7/16" - 6 7/8"	6 7/8" - 7 7/16"
7	6 7/8" - 7 9/16"	7 7/16" - 7 3/4"

Size	Width in mm	Length in mm
Pony	78-85mm	85-92mm
000	86-97mm	93-105mm
00	98-110mm	106-117mm
0	111-116mm	118-124mm
1	117-123mm	125-130mm
2	124-130mm	131-140mm
3	131-139mm	141-150mm
4	140-150mm	151-163mm
5	151-163mm	164-174mm
6	164-175mm	175-188mm
7	176-192mm	189-195mm



## Easyboot LC (Love Child)

The Easyboot LC is a hybrid of the Easyboot Glue-On shell and the EasyShoe Performance. The keyhole design allows for independent heel movement both horizontally and vertically with each foot fall. In addition the flexible rubber insole will allow for natural heel growth and expansion with time. It can be common to see 4-8mm of heel expansion in each four to six week application. The length of the LC is longer than other EasyCare models and should be cut to fit. Cuff height has two lines that guide trimming the upper if a lower profile install is desired. Four holes in the cuff act as dowel pins and strengthen the adhesive bond. The Easyboot LC is intended to be used as a glue-on hoof protection and to be worn throughout the entire trim cycle.



Size Chart – Normal

Size	Width in mm	Length in mm
0	104-107mm	111-118mm
0.5	108-111mm	116-123mm
1	112-115mm	121-128mm
1.5	116-119mm	126-133mm
2	120-123mm	132-139mm
2.5	124-127mm	134-141mm
3	128-131mm	139-146mm
3.5	132-135mm	142-149mm
4	136-139mm	145-152mm

*Always double check the fit using the Fit Kit before purchasing boots. Contact us for more details. 800.447.8836*

When applying the Easyboot LC for a full trim cycle we recommend pretreating the sole with an antimicrobial hoof putty to help prevent the occurrence of thrush.



## Easyboot Glue-On

The Easyboot Glue-On is a high performance boot and an option for horses in speed and long distance events. Although there are more steps to the application process, these boots are perfect for pack trips, speed events like barrel racing, long-distance and multi-day competitions.



In order for this boot to work properly, it must fit snugly. The foot must be dry during application and the glue-on process must be carefully followed. Complete application details are available online.

Success of the Easyboot Glue-On is determined by accurate measurements and a snug fit. For the Easyboot Glue-On, you will need to measure in millimeters. Measuring in inches and converting to millimeters will render the measurement process inaccurate.

Size Chart – Normal

Size	Width in mm	Length in mm
00	96-99mm	106-109mm
00.5	100-103mm	110-113mm
0	104-107mm	114-117mm
0.5	108-111mm	118-121mm
1	112-115mm	122-125mm
1.5	116-119mm	126-129mm
2	120-123mm	130-133mm
2.5	124-127mm	134-137mm
3	128-131mm	138-141mm
3.5	132-135mm	142-145mm
4	136-139mm	146-149mm
4.5	140-143mm	150-153mm
5	144-147mm	154-157mm

Size Chart – WIDE

Size	Width in mm	Length in mm
0W	114-117mm	114-117mm
0.5W	118-121mm	118-121mm
1W	122-125mm	122-125mm
1.5W	126-129mm	126-129mm
2W	130-133mm	130-133mm
2.5W	134-137mm	134-137mm
3W	138-141mm	138-141mm

*Always double check the fit using the Fit Kit before purchasing boots. Contact us for more details. **800.447.8836***

## Easyboot Flip Flop

This new glue on design is a hybrid between the Easyboot Glue-On and the EasyShoe. The Easyboot Flip Flop boasts a simple gluing procedure combined with an open back design allowing it to stay on the hoof for the duration of the trim cycle.



The Easyboot Flip Flop is glued only to the dorsal wall of the hoof. Due to the lack of movement in this part of the hoof, minimal stress is placed on the adhesive bond. This design ensures a reliable bond, week after week.

The open back allows for airflow to the frog and sole and will not hamper the natural mechanics of the back portion of the hoof. The long-term effects of increased hoof flexion can promote a highly developed vascular system along with an overall healthier hoof.

The Easyboot Flip Flop is an easy “set it and forget it” solution well-suited for both performance and therapeutic applications.

Size	Width in mm	Length in mm
0	104-107mm	Trim to size
0.5	108-111mm	Trim to size
1	112-115mm	Trim to size
1.5	116-119mm	Trim to size
2	120-123mm	Trim to size
2.5	124-127mm	Trim to size
3	128-131mm	Trim to size

*Always double check the fit using the Fit Kit before purchasing boots. Contact us for more details. **800.447.8836***

Easyboot Back Country

The Easyboot Back Country takes the best features of the Easyboot Glove and combines them with the Easyboot Trail. The Back Country concept pairs the form-fitting Glove chassis with an easy-to-secure upper.



The Easyboot Back Country features include:

- The Comfort Cup Gaiter is incorporated into the rear of the boot for the latest technology in support and protection of the soft tissue areas.
- The lightweight upper stabilizes the boot on the hoof. It’s quick and easy to put on and take off. No strength or force is required.
- Completely convertible and replaceable; built on the Easyboot Glove chassis. The Easyboot Glove or Easyboot Glue-On can be converted into the Easyboot Back Country (additional accessories required). All components of the Easyboot Back Country are replaceable.

You will need to measure in millimeters for the Easyboot Back Country. Measuring in inches and converting to millimeters will render the measurement process inaccurate. After taking your horse’s measurements, we recommend you order a Fit Kit to ensure you select the correct size for your horse. \*Do not assume that if you wear a certain size in another boot style, you will wear the same size in the Easyboot Back Country.

Size Chart – Normal

Size	Width in mm	Length in mm
00	96-99mm	106-109mm
0.5	100-103mm	110-113mm
0	104-107mm	114-117mm
0.5	108-111mm	118-121mm
1	112-115mm	122-125mm
1.5	116-119mm	126-129mm
2	120-123mm	130-133mm
2.5	124-127mm	134-137mm
3	128-131mm	138-141mm
3.5	132-135mm	142-145mm
4	136-139mm	146-149mm
4.5	140-143mm	150-153mm
5	144-147mm	154-157mm

Size Chart – WIDE

Size	Width in mm	Length in mm
0W	114-117mm	114-117mm
0.5W	118-121mm	118-121mm
1W	122-125mm	122-125mm
1.5W	126-129mm	126-129mm
2W	130-133mm	130-133mm
2.5W	134-137mm	134-137mm
3W	138-141mm	138-141mm

Always double check the fit using the Fit Kit before purchasing boots. Contact us for more details. 800.447.8836

Easyboot Trail

The Easyboot Trail is the easiest hoof boot in the world to apply and remove. The boot opens up completely to easily slip on and off over most hoof shapes and sizes. The rear double Velcro® attachment protects the entire hoof wall and keeps the boot firmly in place. This is the perfect hoof boot for the casual rider.



The Easyboot Trail features include:

- Integrated rear strap adds dependability.
- One of the most affordable boots on the market today.
- Tough polyurethane sole offers excellent durability and grip on all terrain.

Size	Width in inches	Length in inches
0	3 15/16" - 4 1/8"	4 1/8" - 4 5/16"
1	4 1/8" - 4 5/16"	4 5/16" - 4 1/2"
2	4 5/16" - 4 1/2"	4 1/2" - 4 3/4"
3	4 1/2" - 4 3/4"	4 3/4" - 4 15/16"
4	4 3/4" - 4 15/16"	4 15/16" - 5 1/8"
5	4 15/16" - 5 1/8"	5 1/8" - 5 5/16"
6	5 1/8" - 5 5/16"	5 5/16" - 5 1/2"
7	5 5/16" - 5 1/2"	5 1/2" - 5 11/16"
8	5 1/2" - 5 11/16"	5 11/16" - 5 15/16"
9	5 11/16" - 5 15/16"	5 15/16" - 6 1/8"
10	5 15/16" - 6 1/8"	6 1/8" - 6 5/16"
11	6 1/8" - 6 1/2"	6 5/16" - 6 11/16"
12	6 1/2" - 6 7/8"	6 11/16" - 7 1/8"

Size	Width in mm	Length in mm
0	100-105mm	105-110mm
1	105-110mm	110-115mm
2	110-115mm	115-120mm
3	115-120mm	120-125mm
4	120-125mm	125-130mm
5	125-130mm	130-135mm
6	130-135mm	135-140mm
7	135-140mm	140-145mm
8	140-145mm	145-150mm
9	145-150mm	150-155mm
10	150-155mm	155-160mm
11	155-165mm	160-170mm
12	165-175mm	170-180mm

## New Mac

Every New Mac horse boot has a unique hi-tech performance outsole and incorporates EasyCare's patented Thermo Plastic Urethane (TPU) compound which minimizes concussion and shortens recovery time for horses with concussion-related injuries. The New Mac is a great option for pleasure riding and can also serve as a therapy boot. **Sold in pairs.**



### The New Mac features include:

- Fastening system opens wide for ease of application.
- New soft collar upper.
- Integrated rear strap adds dependability.
- Seamless internal webbing strap increases dependability in the most challenging trail conditions.

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
0	3 15/16" - 4 1/8"	4 1/8" - 4 5/16"	0	100-105mm	105-110mm
1	4 1/8" - 4 5/16"	4 5/16" - 4 1/2"	1	105-110mm	110-115mm
2	4 5/16" - 4 1/2"	4 1/2" - 4 3/4"	2	110-115mm	115-120mm
3	4 1/2" - 4 3/4"	4 3/4" - 4 15/16"	3	115-120mm	120-125mm
4	4 3/4" - 4 15/16"	4 15/16" - 5 1/8"	4	120-125mm	125-130mm
5	4 15/16" - 5 1/8"	5 1/8" - 5 5/16"	5	125-130mm	130-135mm
6	5 1/8" - 5 5/16"	5 5/16" - 5 1/2"	6	130-135mm	135-140mm
7	5 5/16" - 5 1/2"	5 1/2" - 5 11/16"	7	135-140mm	140-145mm
8	5 1/2" - 5 11/16"	5 11/16" - 5 15/16"	8	140-145mm	145-150mm
9	5 11/16" - 5 15/16"	5 15/16" - 6 1/8"	9	145-150mm	150-155mm
10	5 15/16" - 6 1/8"	6 1/8" - 6 5/16"	10	150-155mm	155-160mm
11	6 1/8" - 6 1/2"	6 5/16" - 6 11/16"	11	155-165mm	160-170mm
12	6 1/2" - 6 7/8"	6 11/16" - 7 1/8"	12	165-175mm	170-180mm

## Old Mac G2

Every Old Mac's G2 horse boot has a unique hi-tech performance outsole and incorporates EasyCare's patented Thermo Plastic Urethane (TPU) compound which minimizes concussion and shortens recovery time for horses with concussion-related injuries. The Old Mac's G2 is a great option for pleasure riding (less than 25 miles per week) and can also serve as a therapy boot. If your horse suffers from any of the following - arthritis, pedal bone fractures, navicular disease, ringbone, laminitis (founder), bruised soles, or scalping from overreaching - the Old Mac's G2 horse boot will benefit your horse. **Sold in pairs.**



Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
0	3 15/16" - 4 1/8"	4 1/8" - 4 5/16"	0	100-105mm	105-110mm
1	4 1/8" - 4 5/16"	4 5/16" - 4 1/2"	1	105-110mm	110-115mm
2	4 5/16" - 4 1/2"	4 1/2" - 4 3/4"	2	110-115mm	115-120mm
3	4 1/2" - 4 3/4"	4 3/4" - 4 15/16"	3	115-120mm	120-125mm
4	4 3/4" - 4 15/16"	4 15/16" - 5 1/8"	4	120-125mm	125-130mm
5	4 15/16" - 5 1/8"	5 1/8" - 5 5/16"	5	125-130mm	130-135mm
6	5 1/8" - 5 5/16"	5 5/16" - 5 1/2"	6	130-135mm	135-140mm
7	5 5/16" - 5 1/2"	5 1/2" - 5 11/16"	7	135-140mm	140-145mm
8	5 1/2" - 5 11/16"	5 11/16" - 5 15/16"	8	140-145mm	145-150mm
9	5 11/16" - 5 15/16"	5 15/16" - 6 1/8"	9	145-150mm	150-155mm
10	5 15/16" - 6 1/8"	6 1/8" - 6 5/16"	10	150-155mm	155-160mm
11	6 1/8" - 6 1/2"	6 5/16" - 6 11/16"	11	155-165mm	160-170mm
12	6 1/2" - 6 7/8"	6 11/16" - 7 1/8"	12	165-175mm	170-180mm

\* Please note that sizing varies for all EasyCare products. Contact us for more details. **800.447.8836**

Original Easyboot

The hoof boot that started it all - proven success since 1970. The entire boot fits below the hairline, therefore eliminating any chance of rubs. The cable and buckle system helps fit even the most challenging hoof shapes and angles.



Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
Pony	3 3/16" - 3 3/8"	3 3/8" - 3 5/8"	Pony	78-85mm	85-92mm
000	3 3/8" - 3 13/16"	3 5/8" - 4 3/16"	000	86-97mm	93-105mm
00	3 13/16" - 4 3/8"	4 3/16" - 4 5/8"	00	98-110mm	106-117mm
0	4 3/8" - 4 9/16"	4 5/8" - 4 15/16"	0	111-116mm	118-124mm
1	4 9/16" - 4 7/8"	4 15/16" - 5 1/8"	1	117-123mm	125-130mm
2	4 7/8" - 5 1/8"	5 1/8" - 5 9/16"	2	124-130mm	131-140mm
3	5 1/8" - 5 1/2"	5 9/16" - 5 15/16"	3	131-139mm	141-150mm
4	5 1/2" - 5 15/16"	5 15/16" - 6 7/16"	4	140-150mm	151-163mm
5	5 15/16" - 6 7/16"	6 7/16" - 6 7/8"	5	151-163mm	164-174mm
6	6 7/16" - 6 7/8"	6 7/8" - 7 7/16"	6	164-175mm	175-188mm
7	6 7/8" - 7 9/16"	7 7/16" - 7 3/4"	7	176-192mm	189-195mm



Easyboot Mini

The Easyboot Mini has been designed specifically for miniature horses, ponies and foals. Quick and easy to apply, this boot is the ultimate option for tiny hooves.



The lightweight urethane blended sole will allow for traction and protection on any surface. The durable and supple upper keeps the boot secure without interfering with the natural gait.

This is the ideal boot for light driving, therapy and providing traction.

The Easyboot Mini is a first-class option for horses who encounter diversity of terrain. The Easyboot Mini will accommodate a Comfort Pad to be used as a therapy boot.

The Easyboot Mini is the smallest molded hoof boot in the world.

The Easyboot Mini features include:

- Easyboot Mini boots are not left/right specific and will fit front or hind hooves.
- Correct size selection will ensure maximum performance and longevity of the boots.
- Give your horse time to adjust to wearing the Easyboot Mini. Start with conservative use to allow for break-in.
- Gradually increase the duration of use.

Size	Width in mm	Length in mm
Mini	44-49mm	49-54mm
Mini 1	50-55mm	55-64mm
Mini 2	56-65mm	65-72mm
Mini 3	66-76mm	73-86mm



Easyboot Stratus (not for riding)

The new Easyboot Stratus is a premium therapeutic hoof boot system designed to give comfort and support to horses with sore feet, abscesses, founder, laminitic stages, stresses of shipping, recovery after workouts or stalling on hard surfaces. It also provides instant and ongoing relief for horses suffering from chronic lameness and general lower limb or hoof problems by aiding movement and reducing recovery time after injury or surgery.



The boot comes with a patent pending adjustable TheraPad system that allows each pad to be custom tailored to the horse and conditions. The adjustable TheraPad can be used without the TheraRods or the TheraRods can be inserted into the pad to provide more or firmer padding. The three densities of TheraRods provide numerous possibilities.



The Easyboot Stratus features include:

- TheraPad conforms to the hoof and provides additional comfort and support to the sole
- Secure fit allows for light turnout. Designed for use with the Easyboot Click System
- Supportive tread reduces loading of the peripheral hoof wall, spreading weight across the entire solar area
- Back folds down for easy installation and removal

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
00	3 1/2 - 3 7/8"	4 " - 4 3/8"	00	89-99 mm	102-112 mm
0	3 7/8 - 4 1/4"	4 3/8" - 4 5/8"	0	99-108 mm	112-118 mm
1	4 1/4 - 4 7/8"	4 5/8" - 5 1/8"	1	108-124 mm	118-131 mm
2	4 7/8 - 5 1/4"	5 1/8" - 5 3/8"	2	124-133 mm	131-137 mm
3	5 1/4 - 5 7/8"	5 3/8" - 5 7/8"	3	133-149 mm	137-150 mm
4	5 7/8 - 6 1/4"	5 7/8" - 6 1/4"	4	149-159 mm	150-159 mm
5	6 1/4 - 6 11/16"	6 1/4" - 6 11/16"	5	159-170 mm	159-170 mm
6	6 11/16 - 6 15/16"	6 11/16 - 6 15/16"	6	170-177 mm	170-177 mm
7	6 15/16 - 7 3/8"	6 15/16" - 7 3/8"	7	177-188 mm	177-188 mm
8	7 3/8 - 7 1/2"	7 3/8" - 7 7/8"	8	188-191 mm	188-201 mm



EasyCare Therapy Click System (not for riding)



The Therapy Click System is a therapeutic aid and a starting point to change breakover and heel elevation. Both the 5 degree and 10 degree bases have been designed with extra depth to give professionals the ability to customize to each unique case. Breakover position and heel height are quickly changed with a rasp or grinder. Add additional height with adhesive products.

The EasyCare Therapy Click System is available for purchase only upon presentation of a veterinarian’s recommendation or directly purchased by a veterinarian. The veterinarian’s recommendation must include the name of the vet, contact information and confirmation in writing of the veterinarians endorsement of the Therapy Click System.

This policy is in place so that the veterinarian and pet owner can discuss the situation and work together to determine the best protocol. This policy increases the likelihood that the Therapy Click System will be used properly based on veterinarian counsel addressing the specific needs of each individual case.

This policy is applicable only to sales in the US.



Therapy Click 10 Degree



Therapy Click 5 Degree



Insert 4 self-tapping screws

The Therapy Click System can be used in conjunction with the following boot models:

- Easyboot Rx
- Easyboot Cloud
- Easyboot Stratus
- Easyboot Flip Flop
- Original Easyboot
- Easyboot Epic
- Easyboot Back Country
- Easyboot Glove
- Easyboot Glue-On

Note: Therapy Click System is not designed to work with wide sizes.

EasyCare Therapy Click System Size Chart

Easyboot Glove, Glue-On & Back Country	OO	OO.5	O	O.5	1
Click Size	B	B	B	C	C
Easyboot Flip Flop	O	O.5	1	1.5	2
Click Size	B	C	C	C	C
Original Easyboot & Easyboot Epic	Pony	OOO	OO	O	1
Click Size	NA	NA	B	C	C
Easyboot Rx	OO	O	1	2	3
Click Size	C	C	C	D	D
Easyboot Cloud & Stratus	OO	O	1	2	3
Click Size	B	C	C	D	D

# Easyboot Cloud (not for riding)

The Easyboot Cloud is a therapeutic hoof boot system to give comfort and support to horses with thin soles, abscesses, founder, laminitic stages, stresses of shipping, recovery after workouts or stalling on hard surfaces. The Easyboot Cloud also provides instant and ongoing relief for horses suffering from chronic lameness and general lower limb or hoof problems by aiding movement and reducing recovery time after injury or surgery.



## The Easyboot Cloud features include:

- Closed-cell comfort pad conforms to the hoof & provides additional comfort and support to the sole.
- Secure fit allows for light turn-out.
- Supportive tread reduces loading of the peripheral hoof wall, spreading weight across the entire solar area.
- Upper folds back and down completely to allow for ease of access and removal.



Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
00	3 1/2 - 3 7/8"	4 " - 4 3/8"	00	89-99 mm	102-112 mm
0	3 7/8 - 4 1/4"	4 3/8" - 4 5/8"	0	99-108 mm	112-118 mm
1	4 1/4 - 4 7/8"	4 5/8" - 5 1/8"	1	108-124 mm	118-131 mm
2	4 7/8 - 5 1/4"	5 1/8" - 5 3/8"	2	124-133 mm	131-137 mm
3	5 1/4 - 5 7/8"	5 3/8" - 5 7/8"	3	133-149 mm	137-150 mm
4	5 7/8 - 6 1/4"	5 7/8" - 6 1/4"	4	149-159 mm	150-159 mm
5	6 1/4 - 6 11/16"	6 1/4" - 6 11/16"	5	159-170 mm	159-170 mm
6	6 11/16 - 6 15/16"	6 11/16 - 6 15/16"	6	170-177 mm	170-177 mm
7	6 15/16 - 7 3/8"	6 15/16" - 7 3/8"	7	177-188 mm	177-188 mm
8	7 3/8 - 7 1/2"	7 3/8" - 7 7/8"	8	188-191 mm	188-201 mm

1.5	2	2.5	3	3.5	4	4.5	5
C	C	D	D	D	D	E	E
2.5	3						
D	D						
2	3	4	5	6	7		
D	D	E	NA	NA	NA		
4	5	6	7				
E	NA	NA	NA				
4	5	6	7	8			
E	E	NA	NA	NA			



Easyboot Rx (not for riding)

A lightweight therapy boot for horses with hoof maladies, those recovering from surgery or horses who stand on hard surfaces for long periods. It provides support and relief for horses suffering from chronic lameness, laminitis, founder, navicular and other lower limb and hoof problems. An excellent choice for horses on stall rest or confined to light turnout. Size up one size if you plan on using bandages or 12mm Comfort Pads with the Easyboot Rx.

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
00	3 13/16" - 4 3/8"	4 3/16" - 4 5/8"	00	98-110mm	106-117mm
0	4 3/8" - 4 9/16"	4 5/8" - 4 15/16"	0	111-116mm	118-124mm
1	4 9/16" - 4 7/8"	4 15/16" - 5 1/8"	1	117-123mm	125-130mm
2	4 7/8" - 5 1/8"	5 1/8" - 5 9/16"	2	124-130mm	131-140mm
3	5 1/8" - 5 1/2"	5 9/16" - 5 15/16"	3	131-139mm	141-150mm
4	5 1/2" - 5 15/16"	5 15/16" - 6 7/16"	4	140-150mm	151-163mm
5	5 15/16" - 6 7/16"	6 7/16" - 6 7/8"	5	151-163mm	164-174mm
6	6 7/16" - 6 7/8"	6 7/8" - 7 7/16"	6	164-175mm	175-188mm
7	6 7/8" - 7 9/16"	7 7/16" - 7 3/4"	7	176-192mm	189-195mm

Easyboot Zip (not for riding)

The Easyboot Zip is a simple slip-on boot designed to protect the hoof. Safe and easy to apply, the Zip is a better alternative to clumsy foot wraps for medicating the hoof or keeping the hoof free from dirt and oils. The removable internal leather sole helps pull moisture from the hoof and can be easily removed for cleaning. Not for turn-out.



The Easyboot Zip features include:

- Protects the hoof from dirt and oils.
- A better alternative to awkward foot wraps and is ideal for medicating the hoof.
- Allows the hoof to breathe and keeps the hoof clean.
- The Easyboot Zip has a removable leather sole insert and is machine washable.



Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
0	3 - 3 13/16"	4 3/16" - 4 5/8"	0	77-97mm	77-97mm
1	3 7/8 - 4 11/16"	4 5/8" - 4 15/16"	1	98-119mm	98-119mm
2	4 3/4 - 5 3/8"	4 15/16" - 5 1/8"	2	120-136mm	120-136mm
3	5 7/16 - 6 3/8"	5 1/8" - 5 9/16"	3	137-162mm	137-162mm
4	6 7/16 - 7 1/16"	5 9/16" - 5 15/16"	4	163-180mm	163-180mm

EasySoaker (not for riding)

Used for soaking, poulticing or keeping bandages clean and dry. EasyCare recommends sizing up when purchasing an EasySoaker. Sizing up one to two sizes allows extra space for bandages and liquids to be used inside the boot. In addition, a larger size makes application and removal quicker and easier.



Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
Small	3 3/4" - 4 1/2"	4 1/4" - 5"	Small	95-115mm	108-127mm
Medium	4 1/2" - 5 1/4"	5" - 5 3/4"	Medium	115-134mm	127-146mm
Large	5 1/4" - 5 3/4"	5 3/4" - 6 1/4"	Large	134-146mm	146-159mm
X-Large	5 3/4" - 6 1/4"	6 1/4" - 6 3/4"	X-Large	146-159mm	159-172mm
XX-Large	6 1/4" - 6 3/4"	6 3/4" - 7 1/4"	XX-Large	159-172mm	172-184mm
XXX-Large	6 3/4" - 7 1/4"	7 1/4" - 7 3/4"	XXX-Large	172-184mm	184-190mm



EasyShoe

EasyCare’s spirit of innovation continues with the revolutionary EasyShoe product lineup.

Enjoy the benefits of a barefoot horse without having to apply and remove boots each time you ride. The EasyShoe allows the hoof to naturally expand and contract, both laterally and vertically. Manufactured with the latest in polyurethane technology, the EasyShoe dramatically reduces concussion on hard surfaces, yet offers maximum durability and longevity. The open sole design assures complete breathability of the sole and frog.

The EasyShoe has already excelled on the flat track and in the endurance, dressage and sport horse disciplines. The EasyShoe may also be used to help offset chronic symptoms of thin hoof walls, sensitive heels, quarter cracks and slow hoof growth.

- Stays on the hoof for the entire trimming cycle.
- The open toe design means you can set break-over anywhere.

*\* Performance N/G only. Can be nailed or glued.*

EasyShoe Accessories

EasyCare recommends the use of Adhere or EasyShoe Bond to apply the EasyShoe. Please thoroughly review the application guides and videos, as these adhesives have different application protocols. Gluing accessories not included.

EasyShoe *PERFORMANCE*

A wide-webbed shoe with integrated frog support. The unique design maintains support on and around the frog without hampering heel expansion and contraction. The frog support opens with the growth in the heel area during the trimming cycle.



Size	Width in mm	Length in mm
0	105-111mm	108-115mm
1	112-119mm	116-123mm
2	120-127mm	124-131mm
3	128-135mm	132-140mm
4	136-143mm	141-149mm
5	144-152mm	150-159mm



EasyShoe *PERFORMANCE N/G*

A wide-webbed shoe with integrated frog support. The unique design maintains support on and around the frog without hampering heel expansion and contraction. The frog support opens with the growth in the heel area during the trimming cycle. Can be nailed or glued.



Size	Width in mm	Length in mm
0	105-111mm	108-115mm
1	112-119mm	116-123mm
2	120-127mm	124-131mm
3	128-135mm	132-140mm
4	136-143mm	141-149mm
5	144-152mm	150-159mm



EasyShoe *SPORT*

A wide-webbed shoe that offers maximum support across the sole. The internal metal spine allows the shoe to be adjusted to many hoof shapes and creates additional structural integrity.



Size	Width in mm	Length in mm
0	100-110mm	114-117mm
0.5	105-115mm	118-121mm
1	110-120mm	122-125mm
1.5	115-125mm	126-129mm
2	120-130mm	130-133mm
2.5	125-135mm	134-137mm
3	130-140mm	138-141mm



EasyShoe *COMPETE*

A thin-webbed shoe for use on the track or on turf. The internal metal spine allows for maximum adjustment to a wide variety of hoof shapes and sizes. Available with steel wearplate at the toe.



Size	Width in mm	Length in mm
0	95-115mm	111-118mm
1	105-125mm	119-127mm
2	115-135mm	128-136mm
3	125-145mm	137-143mm

# EasyShoe **FLEX**

The newest addition to our EasyShoe line-up is a hybrid nail-on shoe that features the concussion absorption powers of polyurethane while our unique spring steel core provides flexibility and durability that encourages the hoof to flex with each foot fall. The Flex is a great option for distance rides, trail horses, and horse working on hard surfaces. The wide web design also makes the Flex ideal for therapy cases. Holds pour-in packing very well.

The EasyShoe Flex line can be applied with nails, glue or casting. The open nailing slots and clear material make nail placement straight forward and are extremely beneficial for achieving an accurate installation.

Unlike traditional steel or aluminum shoes, no hammer is used to shape the shoe. Simply trace the hoof on the shoe and remove the excess material, as shown in the Flex Application Guide and Video.

The Flex is available in four versions: open heel, heart bar, and full heart bar. Each version is available with a toe clip or side clips. There is also a Flex Light available in the heart bar version without the spring steel core or clips.

**EasyShoe Flex Open Heel:** Open heel version, toe clip & side clip options.



**EasyShoe Flex Heart Bar:** Heart bar version, toe clip & side clip options.



**EasyShoe Flex Full Heart Bar:** Full spring steel heart bar version, toe clip and side clip options. The most rigid option in the Flex line.



## EasyShoe Flex Light

A heart bar shoe that features the concussion absorption powers of polyurethane. The EasyShoe Flex Light does not have the spring steel insert. Can be nailed, glued or cast in place. Very lightweight and long-lasting.



Size	Width in mm	Max Length in mm
0	100-113mm	118mm
1	106-121mm	126mm
2	110-129mm	134mm
3	116-137mm	142mm
4	122-143mm	148mm
5	124-148mm	154mm

# OTHER PRODUCTS

## Stowaway Saddle Packs

Our Stowaway line is the most advanced innovation in “no-bounce” saddle pack design. Maximum storage with minimum bulk, these packs accommodate every riding style. Several compartments allow for better organization and quick, easy access to what you need, when you need it. Superior fabric and heavy-duty adjustment systems make these packs the most durable, user-friendly pack you’ll ever own.



red



hunter green



navy



royal blue



purple



brown



black



colors are approximate

### All Stowaway Packs feature:

- 420D Ripstop with waterproof urethane backing.
- Foam-padded bottom panel lined with durable, soft, short-pile fleece to protect your saddle and your horse from the weight of the pack's contents.
- Extended nylon-tabbed zipper pulls for easy access to all compartments.
- All outer seams are reinforced with durable binding.
- Water resistant material.



#### Deluxe Stowaway

- Made to fit most Western, English or Endurance style saddles.
- 9 zippered easy-access compartments to organize and store essentials.
- Ability to carry up to four #2 Easyboots, Epics or Gloves or two #8 Trails or extra water bottles.
- Nylon adjustable straps with reinforced tabbed ends for tying down a coat or securing contents.
- Large compartments are designed to hold all of your on-the-go necessities.
- Side pouches can hold extra water bottles or horse boots.



#### English Stowaway

- Made to fit any English or English Endurance saddle.
- 5 zippered compartments to organize and store essentials.
- 2 contoured and insulated water bottle holders.
- Comes with two 21 oz. water bottles.
- English Stowaway has special cantle sleeve designed to help keep the pack off the horse's back.
- Nylon adjustable straps with reinforced tabbed ends for tying down a coat or securing contents.



#### Western Stowaway

- Made to fit any Western or Western Endurance saddle.
- 5 zippered compartments to organize and store essentials.
- 2 contoured and insulated water bottle holders.
- Comes with two 21 oz. water bottles.
- Nylon adjustable straps with reinforced tabbed ends for tying down a coat or securing contents.







### Western Stowaway Slim

- Made to fit any Western or Western Endurance saddle.
- 3 zippered compartments to organize and store essentials.
- Same materials and construction features as the regular Western Stowaway pack.
- Nylon adjustable straps with reinforced tabbed ends for tying down a coat or securing contents.



### Water Bottle Stowaway

- Most durable, versatile water bottle holder on the market.
- Comes with two 21 oz. water bottle.
- Bottle strap allows easy access to the bottle while keeping it in place when not in use.
- Attaches to any saddle with a versatile clip and strap system.



### Horse Boot Stowaway

- Will hold up to two size #2 Easyboots, Epics or Gloves or one #8 Trail.
- Two extra compartments for accessories.
- Clip-on top with tie strings to attach anywhere on the saddle.



### Stowaway Pommel

- Made to fit most Western, English and Endurance style saddles.
- 7 zippered compartments to organize and store essentials.
- Center pouch features a cell phone pocket, elastic loop straps, and a sunglasses or rider card compartment.
- Nylon adjustable straps with reinforced tabbed ends help anchor pack and secure contents.



### Deluxe Stowaway Pommel

- Made to fit most Western, English and Endurance style saddles.
- 7 zippered compartments to organize and store essentials.
- 2 contoured and insulated water bottle holders.
- Comes with two 21 oz. water bottles.
- Center pouch features a cell phone pocket, elastic loop straps, and a sunglasses or rider card compartment.
- Nylon adjustable straps with reinforced tabbed ends help anchor pack and secure contents.



# OTHER PRODUCTS

## Stowaway Gear

Our Stowaway hay and gear bags are designed to store, transport and protect supplies like hay, blankets, tools, and extra hoof boots. Stowaway products are made with high quality materials that will hold up and protect against the elements.



### Deluxe Hay/Gear Bag

Perfect for the one-man team, this hay/gear bag will hold up to two regular flakes of hay as well as all the necessities the rider may need. This 6-compartment bag is made with waterproof denier with heavy-duty hardware and zippers. Nylon carrying/hanging straps are removable and adjustable. The hay opening (7" x 7.5") for the horse has a zip-close cover for mess-free transportation.

**Bag Dimensions: Main pouches (combined): 24"W x 19"H x 8"D**

**Smaller side pouch: 24"W x 7.5"H x 5"D**



### Deluxe Full Bale Bag

Will hold a large 3-strand bale. Sturdy nylon web handles, along with durable and waterproof vinyl, make this bag the best on the market. Protect your bales from moisture or greedy horses. 47"L x 24"W x 17"H.



### Deluxe Half Bale Bag

When a full bale is not necessary but a couple flakes aren't enough, this half bale bag is the perfect size for travel. Made with durable and waterproof vinyl, this bag will hold approximately six large flakes of hay. Makes a great large gear bag. 24"L x 24"W x 17"H.



### Deluxe Vinyl Hay Feeder

Made with durable and waterproof vinyl, this hay tote will fit up to two large flakes of hay. The top rim is rigid for easier access and filling. Hanging strap is made with heavy-duty nylon strapping and is detachable on one side for easy hanging on just about anything. 27"H x 19"W x 6"D.



### EasyCare Gear Bag

Made from tough 600D nylon cordura with water resistant PVC backing. Heavy duty zipper and nylon-webbed handles for easy access and travel. Has breathable mesh sides for ventilation. Perfect for holding EasyCare hoof boots, carrying all of your grooming aids and horse blanket storage. 20"L x 15" H x 8" W.

Stowaway Bale and Gear bags are available in **Navy only**.



### Easy Bun Warmer



### Stowaway Utility Belt



### Stowaway Neck Ditty

## E-Z Ride Ultimate Stirrups

E-Z Ride Ultimate Stirrups offer cutting edge technology in a lightweight aluminum hoop design. The integrated top bar system allows for three sizes of fenders and stirrup leathers without the need for additional parts or accessories. The new EVA foam pad cushions the ride and the patented dimple design keeps the rider's foot firmly in place no matter what the riding activity or the rider's footwear. E-Z Ride Ultimate Ultra is equipped with an aluminum base and loop. The E-Z Ride Ultimate is equipped with a black ABS plastic base and aluminum hoop.

**Sold in pairs.**



**E-Z Ride Ultimate Ultra**

Colors: Black or Silver  
Top bar sizes: 1", 2" and 3"



**E-Z Ride Ultimate Ultra with Cage**

Colors: Silver or Black  
Top bar sizes: 1", 2" and 3"



**E-Z Ride Ultimate**

Colors: Silver  
Top bar sizes: 1", 2" and 3"  
Also available with cage.

## E-Z Ride Stirrups

E-Z Ride Stirrups are the #1 long distance stirrup. These nylon stirrups rate the highest in comfort and quality. They are five inches wide from side to side and four inches deep from toe to heel, allowing for the best possible weight distribution throughout the foot. The thick EVA Foam pad provides comfort for the knees and back and helps prevent overall fatigue. The patented top bar deadens the shock transmitted from the horse and long hours in the saddle without excessive bounce. Stirrups come in several different top bar sizes to fit your type of saddle. Optional nylon safety cages are available.

Nylon E-Z Ride Stirrups are lighter weight and less expensive than metal stirrups. These stirrups have all the same great qualities and are fast becoming the most popular endurance stirrup on the market. Made with a heavy duty, hi-tech polymer. Optional safety cages are available.

**Sold in pairs.**



**Nylon Standard**

Colors: Black or Brown.  
Top bar sizes: 1.5", 2", 2.5" or 3"



**Nylon with Cage**

Colors: Black or Brown  
Top bar sizes: 1.5", 2", 2.5" or 3"



**Standard Nylon with Leather Tapadero Cage**

Colors: Black or Brown  
Top bar sizes: 1.5", 2", 2.5" or 3"

## OTHER PRODUCTS

### The HiTie™ Trailer Tie System

---

The HiTie was developed to provide a comfortable and safe alternative to trailer tying and portable corrals. The HiTie gives horses free movement and access to an area equivalent to a 13 foot diameter round pen. This safe, controlled movement gives horses the ability to graze, eat, drink, roll and lie down while tied.

#### The HiTie System

---



Heavy duty stainless steel and high strength fiberglass construction safely secures your horse. Conveniently stows flush against your trailer when not in use so there's never a need to remove it.

#### HiTie System Package includes:

- HiTie arm
- Mounting hardware with instructions
- D-Ring Pin & Safety Clip



#### Important Installation Information:

---

- Installation instructions are intended as a guide only. It is your responsibility to determine the proper location on your particular trailer to mount the HiTie Trailer Tie System.
- As a general, common sense rule, the HiTie System MUST be mounted to an area on the trailer wall that can support both the weight of the system and any loads put on it by the horse. Mounting to an area with an internal support beam is ideal.
- EasyCare assumes no responsibility for damages or injuries caused by improper mounting.





## HiTie Accessories Below Sold Separately



### HiTie Bungee

All-in-one adjustable length strap attached to a heavy duty elastic cord. System includes everything you need to tether your horse to the HiTie Arm. Bridle clip on one end, hook and loop system on the other end. Designed to break free under heavy stress for horse safety. Black only.



### The Clip

2-part system allows use of a standard lead rope (not included) instead of the Bungee Tether to secure your horse to the HiTie Arm.

The Clip is heavy-duty all metal construction. Spring clip on one end, eye, guide and adjustable tensioner for fastening a lead rope on the other end.



### D-Ring Pin & Safety Clip

The heavy-duty D-Ring Pin and Safety Clip keeps the HiTie Arm securely in place when in use or stowed for travel or storage.

The HiTie System includes one D-Ring and safety clip.

Order as a spare or replacement.



### The Clip Quiet

The Clip Quiet connects the Clip to the HiTie arm. Heavy duty webbing construction with a hook and loop fastening system.

Reduces wear and stops rattling noises by eliminating metal-on-metal contact. Provides break protection for horse safety.



# EasyCare Inc.

## How to Contact Us:

2300 E. Vistoso Commerce Loop Rd. Tucson, AZ 85755

Phone: 800.447.8836 • Fax: 520.297.9600

**admin@easycareinc.com**

**easycareinc.com**

 **/easyboot**

 **/easycarevideos**

Contact us or  
visit our website  
to find a dealer  
near you.

*Designed and tested in  
Durango, Colorado USA*



**EasyCare Inc.**

**Est. 1970**