

Easyboot Cloud Application Guide


The Easyboot Cloud is a therapeutic hoof boot system to give comfort and support to horses with thin soles, abscesses, founder, laminitic stages, stresses of shipping, recovery after workouts or stalling on hard surfaces. The Easyboot Cloud also provides instant and ongoing relief for horses suffering from chronic lameness and general lower limb or hoof problems by aiding movement and reducing recovery time after injury or surgery.

The supportive sole of the Easyboot Cloud is designed to allow the horse to stand more comfortably by reducing pressure on the sensitive areas of the hoof and lower leg. The increased comfort level stimulates blood flow and a faster recovery time from many lower leg pathologies. At its base is a trail-proven high traction tread pattern that is proven in multiple disciplines.

Each Easyboot Cloud comes with a removable closed-cell EVA comfort pad to provide unequalled relief within the boot. Replacement EVA comfort pads are sold separately. The Easyboot Cloud may be used in conjunction with the EasyCare Therapy Click System.®

Seams and edges of the industrial strength Cordura® upper are rounded for safety. A padded collar provides additional relief to the pastern area. Three air vents provide air circulation to keep the hoof cool while preventing the accumulation of dirt and debris.

The unique upper of the Easyboot Cloud folds down completely at the rear to accommodate the widest hooves and allow for quick application without strength or force. After the hoof is placed in the Easyboot Cloud, two tabs with a low profile industrial strength hook and loop system provide a fail-proof and dependable fit. The double hook and loop attachment tabs prevent the boot from opening unintentionally.

 *The Easyboot Cloud. Instant mobility for horses in pain. It's like walking on a cloud.*

The Easyboot Cloud is available in ten sizes that accommodate many hoof shapes and angles.



EVA Comfort Pad

conforms to the hoof and provides additional comfort and support to the sole.



Secure Fit

allows for light turn-out. May also be used with the EasyCare Therapy Click System.



Supportive Tread

reduces loading of the peripheral hoof wall, spreading weight across the entire solar area.

Applying The Easyboot Cloud

1. Loosen the hook and loop closure at the top of the boot and bend the rear of the boot collar back.
2. Pick up the hoof. Hold the boot from the bottom and slip it over the toe.
3. Push the boot until the toe meets the front of the boot. Do not pull on the upper material.
4. Place the hoof on the ground to set the heel and return the collar to the upright position.
5. Fasten the hook and loop closure at the top of the boot.

Removing the Easyboot Cloud

1. Unfasten the hook and loop at the top of the boot.
2. Pull the boot off by pulling down and away on the back of the boot.

Helpful Hints

Each Easyboot Cloud comes with a removable EVA Comfort Pad. Replacement comfort pads are sold separately. There is a corresponding pad for each boot size.

Allow your horse time to become accustomed to wearing the Easyboot Cloud hoof boots.

Take hoof measurements immediately following a fresh trim. Follow the steps on the back of this guide to select the correct size. Feel free to contact a member of our customer service team for assistance.

Please consult the size chart for sizing the Easyboot Cloud for your horse. Easyboot Cloud sizing allows for natural hoof growth between trims. If you intend to use Easyboot Cloud boots over metal shoes*, take your hoof measurements after the horse is freshly shod.

**Using our hoof boots over metal shoes voids our 90-day warranty.*

EasyCare Inc.

30-Day Money-Back Guarantee

We back all of our products with a money-back guarantee. If you are not completely satisfied with a product for any reason, you may return it for a refund of the purchase price (excluding shipping and handling) within 30 days of purchase. To qualify, all products must be returned to place of purchase and have a dated receipt. Offer not valid for EasyCare bargain bin or factory seconds. If the product was purchased from EasyCare, contact us at 800-447-8836 to receive return authorization and an RMA number. All returns must have an RMA number clearly written on the outside of package. Returns without an RMA number will be refused.

EasyCare Inc.

90-Day Warranty

EasyCare guarantees the quality and workmanship of all EasyCare products for 90 days from date of purchase. While this warranty is very broad in nature, it does not cover wear (including sole breakthrough when worn over horseshoes), misuse, abuse, factory seconds, bargain bin items, or boots purchased as used. Products returned for evaluation must be washed clean, dry and free of debris, dirt, sand and manure. Damage inflicted by omnivorous horses, rodents or other unusual abuse is not covered. Customers in the United States should contact EasyCare directly for all warranty issues at 800-447-8836. International customers should contact the dealer where the product was purchased.

Join The Conversation

For up to the minute tips & tricks follow EasyCare:



Facebook
facebook.com/easyboot




The EasyCare Blog
blog.easycareinc.com

All products should be used in accordance with the instructions in this application guide. EasyCare Inc. hereby disclaims responsibility for the misuse of any product and for losses incurred through the use of any product described in this guide, on EasyCare Inc. product packaging, or on easycareinc.com. EasyCare Inc. products are not intended to diagnose or cure any medical condition.

easycareinc.com 800.447.8836

How to Measure for Boots

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

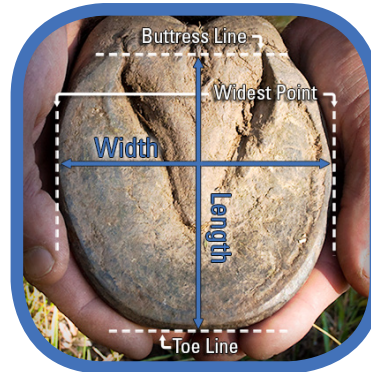

3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

Size	Width in inches	Length in inches
00	3 1/2" - 3 7/8"	4" - 4 3/8"
0	3 7/8" - 4 1/4"	4 3/8" - 4 5/8"
1	4 1/4" - 4 7/8"	4 5/8" - 5 1/8"
2	4 7/8" - 5 1/4"	5 1/8" - 5 3/8"
3	5 1/4" - 5 7/8"	5 3/8" - 5 7/8"
4	5 7/8" - 6 1/4"	5 7/8" - 6 1/4"
5	6 1/4" - 6 11/16"	6 1/4" - 6 11/16"
6	6 11/16" - 6 15/16"	6 11/16" - 6 15/16"
7	6 15/16" - 7 3/8"	6 15/16" - 7 3/8"
8	7 3/8" - 7 1/2"	7 3/8" - 7 7/8"

Size	Width in mm	Length in mm
00	89-99 mm	102-112 mm
0	99-108 mm	112-118 mm
1	108-124 mm	118-131 mm
2	124-133 mm	131-137 mm
3	133-149 mm	137-150 mm
4	149-159 mm	150-159 mm
5	159-170 mm	159-170 mm
6	170-177 mm	170-177 mm
7	177-188 mm	177-188 mm
8	188-191 mm	188-201 mm

APPLICATION GUIDE

Easyboot® CLOUD

