

EASYSOAKER APPLICATION

EasySoaker Applications

The EasySoaker is perfect for a variety of applications where temporary protection or treatment of the hoof is required. As with any hoof or leg related injury, consult your veterinarian or hoof care professional for diagnosis and treatment recommendations.

Navicular:

- Reduced concussion
- Keep foot moist through use of conditioners inside boot
- Complete protection of painful areas
- Cold water soaking

Founder:

- Poultice applications
- Warm water soaks
- Keep foot clean and pliable

Abscesses:

- Applications of germicide and poultices
- Pad inside boot to hold medication at abscess site
- Keep foot clean and free from dirt and bacteria
- Soak in Epsom salts

Thrush:

- Protection while infection drains
- Pad and pack foot with medication
- Keep foot clean and free from dirt and bacteria

Sand Cracks:

- Overall support for cracked hoof to allow growth
- Protect sensitive areas (with packing or padding) during growth period

Seedy Toe:

- Applications of bacterial agent
- Protection of tender areas

Contracted Heels:

- Keep foot moist and pliable through use of hoof conditioner
- Restore moisture through use of poultices or clay packs
- Allow natural heel expansion and hoof protection while barefoot

Corns:

- Provide protection and support for the hoof while leaving barefoot
- Pad or boot to reduce concussion
- Protect draining areas with pad
- Hold medication at site of wound with pad or boot

White Line Disease:

- Application of fungicide and bactericide by use of pad or boot
- Protect affected areas from further invasion from dirt and debris
- Reduce chance of future injury due to bruising or quicking
- Protect hoof wall from excessive chipping during growth period

Fitting the EasySoaker on Your Horse

1. Loosen the webbing and hook and loop closure at the top of the boot, allowing the strap to move freely.
2. Slip the boot over the toe and use the pull tabs located at the top of the boot to pull boot on.
3. Place the hoof on the ground to set the heel. Secure by tightening the strap. Secure the strap with hook and loop closure after tightening.



1.



2.



2.



3.

To Remove the EasySoaker

1. Unfasten the hook and loop to release the strap. Pull the boot off by pulling down and away on the back of the boot.



1.



EasyCare Inc.
The Ultimate in Hoof Protection

HOW TO MEASURE FOR THE EASYBOOT TRAIL

EasySoaker

APPLICATION GUIDE

The EasySoaker is packed with features

- » Textured non-slip base provides traction.
- » Secures medication with no risk of tipping over.
- » Easy-open buckle makes application and removal simple.
- » Hook and loop fastener secures the top.

Accessories

Easy Dirt Skirt

Light sheet foam which fits under the hoof and extends up the back of the heel. Helpful for holding medication at wound site.



Comfort Pads & Dome Comfort Pads

Check out EasyCare's line of Comfort Pads to provide extra cushioning and support. (We recommend that you work with your veterinarian or hoof care practitioner when treating any type of hoof ailment.)



1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

To determine the proper size boot for your horse, measure the width of the affected hoof, across the bottom at the widest point and from toe to buttress line. **We recommend sizing up when purchasing an EasySoaker.** This allows extra space for bandages and liquids to be used inside the boot. A larger size also makes application and removal quicker and easier.



Join the Conversation

For tips and tricks follow EasyCare:



A product of EasyCare Inc.

EasyCare Guarantee

800.447.8836 | easycareinc.com

All products should be used in accordance with the instructions in this application guide. EasyCare Inc. hereby disclaims responsibility for the misuse of any product and for losses incurred through the use of any product described in this guide, on EasyCare Inc. product packaging, or on easycareinc.com. EasyCare Inc. products are not intended to diagnose or cure any medical condition.

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
Small	3 3/4" - 4 1/2"	4 1/4" - 5"	Small	95 - 115mm	108 - 127mm
Medium	4 1/2" - 5 1/4"	5" - 5 3/4"	Medium	115 - 134mm	127 - 146mm
Large	5 1/4" - 5 3/4"	5 3/4" - 6 1/4"	Large	134 - 146mm	146 - 159mm
X-Large	5 3/4" - 6 1/4"	6 1/4" - 6 3/4"	X-Large	146 - 159mm	159 - 172mm
XX-Large	6 1/4" - 6 3/4"	6 3/4" - 7 1/4"	XX-Large	159 - 172mm	172 - 184mm
XXX-Large	6 3/4" - 7 1/4"	7 1/4" - 7 3/4"	XXX-Large	172 - 184mm	184 - 190mm



EasyCare Inc.
The Ultimate in Hoof Protection