

EASYSHOE FIT KIT INSTRUCTIONS

The EasyShoe Fit Kit

The ideal tool for determining proper sizing for the EasyShoe product line.

This Fit Kit contains:

1. Three EasyShoes.



2. A postage paid return label and envelope.

3. Fitting instructions and photos.

This complete Fit Kit must be returned in good, clean condition within 30 days of ordering. Failure to do so will result in a charge to you of the retail price, plus a non-return fee.

The new EasyShoe allows the hoof to expand and contract laterally and vertically. They are lightweight and simple, and can be glued or nailed* to the hoof. The EasyShoe stays on the hoof for the entire trimming cycle. Ideal for performance, pleasure or therapeutic applications, the open toe design accommodates a wide variety of hoof shapes and sizes.




The EasyShoe Fit Kit will allow you to properly determine if the EasyShoe model you have chosen will fit your horse's feet well, and exactly which size fits best.

*EasyShoe Performance N/G only.

Before receiving this kit, you should have taken accurate measurements of your horse's clean hooves after a fresh trim. You are receiving the shoe sizes your horse should fit into, based on your measurements, plus one size above, and one size below.

How the Fit Kit Works

1. Measure your horse's feet (see reverse for detailed measuring instructions).
2. Compare your measurements with the appropriate EasyShoe size charts.

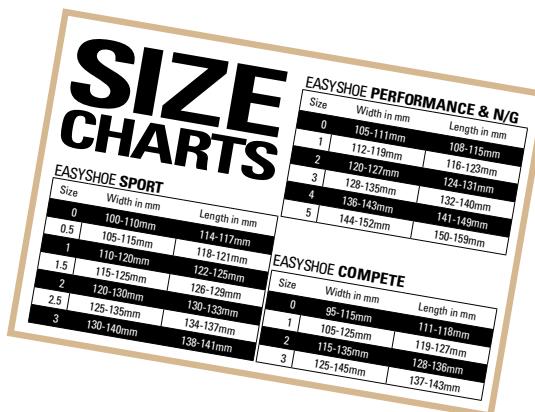

3. Your measurements will determine which size Fit Kit you need. For example, if you have selected the EasyShoe Sport as your chosen model, and your horse's hoof measurements call for a size #1.5 EasyShoe Sport, order a #1.5 EasyShoe Sport Fit Kit. It will come with the following sizes of single EasyShoe Sport shoes: #1, #1.5 and #2.

Keep in mind that most horses require larger sizes on the front feet, so you may need to order two EasyShoe Fit Kits: one kit to accommodate the front feet and one to accommodate the rear feet.

4. You will be charged a flat postage fee per Fit Kit for delivery, which includes a prepaid return postage label and envelope.

5. When the shoes arrive, try them on the horse's feet to determine the optimum EasyShoe size:
 - a. The optimum size shoe can be pushed on the hoof and will provide a snug fit at the glue wings, with virtually no gaps.
 - b. The EasyShoe will provide enough support at the back of the hoof when setting breakover at the desired location.
6. Record the proper shoe size.
7. If your chosen EasyShoe style does not fit well, you may want to consider an alternative EasyShoe style or one of our EasyCare hoof boots.
8. If the EasyShoes become dirty or muddy, please rinse them off and dry them thoroughly to avoid mold during shipping and as a courtesy to our staff.
9. Return the EasyShoe Fit Kit to EasyCare in the addressed, postage paid envelope.
10. For tips, instructional videos or to order the proper EasyShoe or EasyCare products, visit easycareinc.com or call 800-447-8836.



This EasyShoe is not large enough for the hoof. It will not provide sufficient heel support.



This EasyShoe is too large for the hoof. The tabs at the side do not provide full contact with the hoof wall.



EasyCare
easycareinc.com

What Should You Look For in a Poor Fit?

1. Gaps (small or large) can be seen between the vertical glue wing and the hoof wall.
2. A lack of support at the rear of the hoof capsule.

What Can You Do to Turn a Bad Fit into a Good Fit?

1. Have your horse trimmed by a hoof care professional. Make sure they acknowledge the mechanical effects of toe and heel length.



2. It is likely that after appropriate trimming by a hoof care professional, you will have success with the EasyShoe Fit Kit.
3. Some feet may not be the right shape for the EasyShoe. If this is the case, please consider other products in the EasyCare product line.

We continue to strive for the best products on the market with you and your horse in mind. We hope you enjoy our product selection. Thank you for your continued patronage.


The Fine Print

EasyCare Inc. hereby disclaims responsibility for the misuse of any products described in this brochure and website and/or losses incurred through the use of products by any method other than that which is outlined in this brochure or web site or EasyCare's product packaging. All products should be used in accordance with the instructions printed on the product label. EasyCare Inc. products are not intended to treat, cure or diagnose any medical condition.

easycareinc.com • 800.447.8836
blog.easycareinc.com • Facebook.com/Easyboot
© Copyright EasyCare, Inc. 2014

How to Measure

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

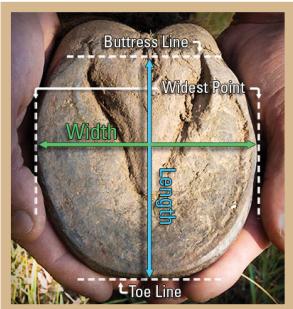

3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this shoe style is not recommended. Be sure to check your measurements against an alternative product in the EasyCare line-another style may accommodate your horse's hoof shape better.

SIZE CHARTS

EASYSHOE SPORT

| Size | Width in mm | Length in mm |
|------|-------------|--------------|
| 0 | 100-110mm | 114-117mm |
| 0.5 | 105-115mm | 118-121mm |
| 1 | 110-120mm | 122-125mm |
| 1.5 | 115-125mm | 126-129mm |
| 2 | 120-130mm | 130-133mm |
| 2.5 | 125-135mm | 134-137mm |
| 3 | 130-140mm | 138-141mm |

EASYSHOE PERFORMANCE & N/G

| Size | Width in mm | Length in mm |
|------|-------------|--------------|
| 0 | 105-111mm | 108-115mm |
| 1 | 112-119mm | 116-123mm |
| 2 | 120-127mm | 124-131mm |
| 3 | 128-135mm | 132-140mm |
| 4 | 136-143mm | 141-149mm |
| 5 | 144-152mm | 150-159mm |

EASYSHOE COMPETE

| Size | Width in mm | Length in mm |
|------|-------------|--------------|
| 0 | 95-115mm | 111-118mm |
| 1 | 105-125mm | 119-127mm |
| 2 | 115-135mm | 128-136mm |
| 3 | 125-145mm | 137-143mm |



EasyShoe PERFORMANCE & N/G



EasyShoe SPORT



EasyShoe COMPETE



EasyShoe PERFORMANCE

EasyShoe

Fit Kit

APPLICATION GUIDE