

EASYBOOT FLIP FLOP FIT KIT



The ideal tool for determining proper sizing for the Easyboot Flip Flop.


This Easyboot Flip Flop Fit Kit contains:

1. Four Easyboot Flip Flops
2. A postage paid return label and envelope
3. Fitting instructions and photos

Your complete fit kit must be returned in good, clean condition within 30 days of your invoice date. Failure to do so will result in a charge to you of the retail price, plus a non-return fee.



Each Easyboot Flip Flop has only four millimeters of width difference between each size. For reference, 4mm is the width of three dimes. This tight gap between sizes helps ensure a snug fit, but it also means there is little margin for error when measuring and determining the proper Easyboot Flip Flop size.

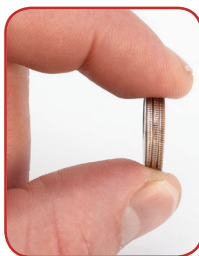

The lightweight Easyboot Flip Flop allows for both lateral and vertical expansion and contraction of the foot and is also simple to apply. The Easyboot Flip Flop can remain on the hoof for the entire trimming cycle. Ideal for performance, pleasure or therapeutic application, the Easyboot Flip Flop can also be heat fitted to accommodate various hoof pathologies.

Before receiving this kit, you should have taken accurate measurements of your horse's hooves in millimeters immediately following a trim.

How the Fit Kit Works

1. Measure the width of your horse's hooves in millimeters.
2. Compare your measurements with the Easyboot Flip Flop size chart.
3. Your measurement will determine which size fit kit you will need.
4. You will be charged a flat postage fee of \$16.00 for fit kit delivery, which includes a prepaid return postage envelope.
5. When the fit kit arrives, try the shells on the horse's hooves to determine the correct size.
 - a. The optimum size Easyboot Flip Flop can be pushed on the hoof and will provide a snug fit at the gluing cuff, with virtually no gaps. You should feel the cuff flex as it goes over the hoof.



- b. The back of the Easyboot Flip Flop may extend past the heel and can be trimmed down once the correct size is determined and purchased. Please do not trim the Easyboot Flip Flop Fit Kit.



6. Record the size of Easyboot Flip Flop you will need to purchase.
7. If the Easyboot Flip Flop does not fit well, you may want to consider alternative EasyCare hoof protection.
8. If the Easyboot Flip Flops become dirty or muddy, please rinse them off and dry them thoroughly to avoid mold during shipping and as courtesy to our staff.
9. Return the Easyboot Flip Flop Fit Kit to EasyCare in the addressed, postage paid envelope.
10. For instructional videos, tips, and to order the Easyboot Flip Flop or other EasyCare products, please visit easycareinc.com or call 800-447-8836.


What Are Indicators of a Poor Fit?

1. Gaps between the vertical gluing cuff and the hoof wall.
2. A lack of support at the rear of the hoof capsule.



How to Measure

1. After a trim, measure in millimeters the width of the hoof across the sole at the widest point. There is no need to measure the length.
2. Compare your measurement with the size chart below.

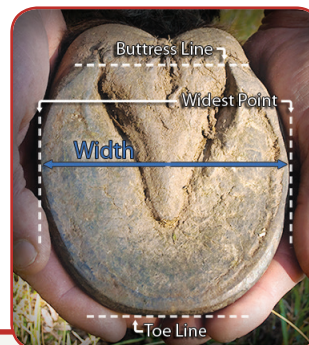


Easyboot Flip Flop Size Chart

Size	Width in mm	Length in mm
1	112-115mm	Trim to size
1.5	116-119mm	Trim to size
2	120-123mm	Trim to size
2.5	124-127mm	Trim to size

Join the Conversation

For tips and tricks follow EasyCare:

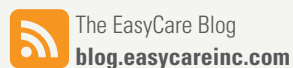

FIT KIT EASYBOOT FLIP FLOP

