

# Easyboot Stratus Application Guide

The Easyboot Stratus is a premium therapeutic hoof boot system designed to give comfort and support to horses with sore feet, abscesses, founder, laminitic stages, stresses of shipping, recovery after workouts or stalling on hard surfaces. It also provides instant and ongoing relief for horses suffering from chronic lameness and general lower limb or hoof problems by aiding movement and reducing recovery time after injury or surgery.

The boot comes with a patent pending adjustable TheraPad system that allows each pad to be custom tailored to the horse and conditions. The adjustable TheraPad can be used without the TheraRods or the TheraRods can be inserted into the pad to provide more or firmer padding. The three densities of TheraRods provide numerous possibilities.


Each Easyboot Stratus comes with a removable closed-cell EVA comfort pad to provide unequalled relief within the boot. Replacement EVA comfort pads are sold separately. The Easyboot Stratus may be used in conjunction with the EasyCare Therapy Click System.®

The Upper is made of premium leather. A padded collar provides additional relief to the pastern area. Internal neoprene is used in the heel area for comfort. A patent pending "heel lock" system hold the heel in place a prevents twisting. Three air vents provide air circulation to keep the hoof cool and prevents the accumulation of dirt and debris.

The Easyboot Stratus will come packaged with boot, TheraPad and 15 TheraRods (5 comfort rods in three densities). Replacement TheraPads and TheraRods are separately. The Easyboot Stratus may be used in conjunction with the EasyCare Therapy Click System®.



*The Easyboot Stratus may be used in conjunction with the EasyCare Therapy Click System®*

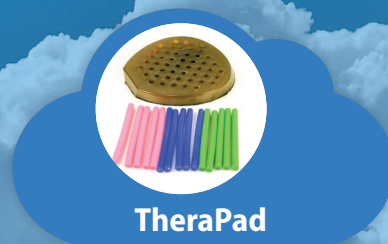

**TheraPad**

the boot comes with a patent pending adjustable TheraPad system that allows each pad to be custom tailored to the horse and conditions.



**Secure Fit**

allows for light turn-out. May also be used with the EasyCare Therapy Click System.



**Supportive Tread**

reduces loading of the peripheral hoof wall, spreading weight across the entire solar area.

## Applying The Easyboot Stratus

1. Loosen the hook and loop closure at the top of the boot and bend the rear of the boot collar back.
2. Pick up the hoof. Hold the boot from the bottom and slip it over the toe.
3. Push the boot until the toe meets the front of the boot. Do not pull on the upper material.
4. Place the hoof on the ground to set the heel and return the collar to the upright position.
5. Fasten the hook and loop closure at the top of the boot.

## Removing the Easyboot Stratus

1. Unfasten the hook and loop at the top of the boot.
2. Pull the boot off by pulling down and away on the back of the boot.

## Helpful Hints

Each Easyboot Stratus comes with a TheraPad and 15 TheraRods. Replacement TheraPad and TheraRods are sold separately. There is a corresponding pad size for each boot size.

Allow your horse time to become accustomed to wearing the Easyboot Stratus hoof boots.

Take hoof measurements immediately following a fresh trim. Follow the steps on the back of this guide to select the correct size. Feel free to contact a member of our customer service team for assistance.

Please consult the size chart for sizing the Easyboot Stratus for your horse. Easyboot Stratus sizing allows for natural hoof growth between trims. If you intend to use Easyboot Stratus boots over metal shoes\*, take your hoof.

*\*Using our hoof boots over metal shoes voids our 90-day warranty.*

## EasyCare Inc.

### 30-Day Money-Back Guarantee

We back all of our products with a money-back guarantee. If you are not completely satisfied with this product for any reason, EasyCare will gladly refund your purchase price (excluding shipping and handling) within 30 days of purchase. To qualify, products must be returned to EasyCare with a completed Return Merchandise Authorization (RMA) form. Products will not be accepted without an RMA number and form. The RMA can be obtained from [www.easycareinc.com](http://www.easycareinc.com) or by calling an EasyCare Product Specialist at 800.447.8836. Customers are responsible for the cost of shipping returned product to EasyCare.

Warranty is void for boots used over steel, aluminum or composite horseshoes, for damage inflicted by omnivorous horses, rodents, or other unusual abuse.

All glue-on boots, shoes, products and accessories must be returned in their original and unopened packaging. EasyCare, at its sole discretion, may void warranty if glue-on products appear to have been removed from packaging.

Bargain Bin, factory seconds, and products purchased used are not covered under any EasyCare warranty.

**If you have purchased our product through one of our partners, then please return the product to the place of purchase for the same warranty service. EasyCare does not accept items purchased from our dealers.**

## EasyCare Inc.

### 90-Day Warranty

EasyCare Inc. offers a repair or replacement warranty on quality and workmanship for all products manufactured under the EasyCare label within 90 days of purchase. Products must be returned to EasyCare with a copy of the customer's dated receipt and a completed Return Merchandise Authorization (RMA) form. Products will not be accepted without a dated proof of purchase and a completed RMA form. The RMA can be obtained from [www.easycareinc.com](http://www.easycareinc.com) or by calling an EasyCare Product Specialist at 800.447.8836.

EasyCare staff is always willing to discuss any specific issues you may have regarding the performance of your EasyCare products. If a boot fails during warranty, it is generally a minor hardware issue. Please bear in mind that most components of EasyCare hoof boots are either replaceable or repairable. This includes wire lace, cables, buckles, rollers, screws, straps and gaiters. Most repair can be easily accomplished with simple household tools. Videos of many repairs are available on our website at [www.easycareinc.com](http://www.easycareinc.com). You may also contact us at 800.447.8836 or [admin@easycareinc.com](mailto:admin@easycareinc.com).

Products returned for evaluation must be clean and dry. Failure to return clean and dry products will result in a cleaning fee. Products that can be repaired will be repaired and returned to the customer promptly. Products that cannot be repaired will be replaced. You are responsible for shipping costs to EasyCare. EasyCare will pay for shipping back to you if product is covered under warranty. If product is not covered under warranty then we will contact you for return shipping payment. If we have not heard back from you within two weeks after contact then we will dispose of returned products.


Warranty is void for boots used over steel, aluminum or composite horseshoes, for damage inflicted by omnivorous horses, rodents, or other unusual abuse.

Bargain Bin, factory seconds, and products purchased used are not covered under any EasyCare warranty.

**[easycareinc.com](http://easycareinc.com) 800.447.8836**

## How to Measure for Boots

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

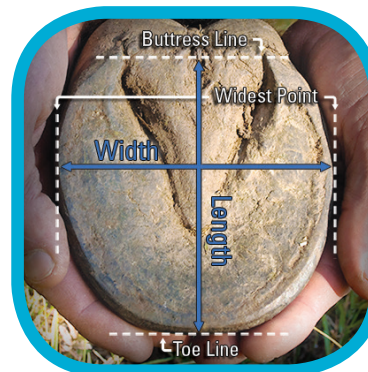

3. Compare your measurement with the size chart below.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

### Join The Conversation

For up to the minute tips & tricks follow EasyCare:



Size	Width in inches	Length in inches
00	3 1/2" - 3 7/8"	4" - 4 3/8"
0	3 7/8" - 4 1/4"	4 3/8" - 4 5/8"
1	4 1/4" - 4 7/8"	4 5/8" - 5 1/8"
2	4 7/8" - 5 1/4"	5 1/8" - 5 3/8"
3	5 1/4" - 5 7/8"	5 3/8" - 5 7/8"
4	5 7/8" - 6 1/4"	5 7/8" - 6 1/4"
5	6 1/4" - 6 11/16"	6 1/4" - 6 11/16"
6	6 11/16" - 6 15/16"	6 11/16" - 6 15/16"
7	6 15/16" - 7 3/8"	6 15/16" - 7 3/8"
8	7 3/8" - 7 1/2"	7 3/8" - 7 7/8"

Size	Width in mm	Length in mm
00	89-99 mm	102-112 mm
0	99-108 mm	112-118 mm
1	108-124 mm	118-131 mm
2	124-133 mm	131-137 mm
3	133-149 mm	137-150 mm
4	149-159 mm	150-159 mm
5	159-170 mm	159-170 mm
6	170-177 mm	170-177 mm
7	177-188 mm	177-188 mm
8	188-191 mm	188-201 mm

## APPLICATION GUIDE

